
ENFit™

LOW DOSE ENTERAL SYRINGE WITH ENFit CONNECTOR FOR ENTERAL USE.

INSTRUCTIONS FOR USE

HOW TO USE ENTERAL LOW DOSE SYRINGES TO AVOID OVERDOSING

The syringe should be filled using a bottle adapter or straw to keep the syringe tip (moat) clear of fluid. You must check the moat is clear of any excess fluid before administration. Tap/flick the syringe until the excess fluid has been fully removed from the moat.

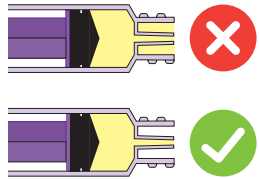STORAGE CONDITIONS

LABELLING SYMBOLS

MEDICINA LTD. Unit 1-4, Rivington View Business Park, Station Road, Bolton BL6 5BN, UK.

ENFit is a registered trademark of GEDSA used with their permission.

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ENFITPELDIFU001IT

INTENDED USE

Enteral syringe with ENFit connector is used for delivery of enteral feeding, Aspiration, flushing and Enteral Medicines.

The syringe is intended for single use on a single patient.

INDICATIONS

The device is indicated for use as a dispenser, a measuring device and a fluid transfer device. It is used to deliver fluids into the body via extension sets and feeding tubes which have ENFit compatible male connectors

INTENDED USER

Medical personnel and or appropriately trained individuals for home use.

DEVICE DESCRIPTION

Enteral syringe with ENFit connector.

CONTRAINDICATIONS

The syringe is not intended for parenteral use.

WARNINGS/ PRECAUTIONS

Do not re-use or reprocess as this may lead to leakage or infection.

Do not use if the packaging is open, syringe tip is visibly blocked, or the syringe is damaged.

Medicina does not recommend the use of the “cupfill” technique for filling syringes.

Should fluid get into the tip of the syringe or the moat of the Low dose syringes then it should be removed by tapping the syringe.

Improper selection of syringe or syringe profile in a pump menu may lead to inaccurate delivery to the patient.

DIRECTIONS

Always follow the hospital protocol for using Enteral syringes.

Syringe must be connected to a male ENFit compatible connector during filling, delivery AND any other plunger operation to reduce the risk of dosing volume inaccuracies.

Fully depress the plunger before filling. Draw liquid into syringe. For accurate measurement use the leading edge of the black gasket. Do not fill beyond the maximum graduation.

To expel air bubbles, hold syringe with tip pointing up. Tap side of syringe to allow bubbles to rise toward tip. Draw back slightly on plunger, the push plunger upwards to eject air in the barrel and tip. Repeat until bubbles are eliminated.

Use immediately or cap off and label syringe until ready to use.

DISPOSAL

Dispose of syringe as required by Hospital Protocols or if used at home dispose of in household rubbish.

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